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
The screenshot shows the 'Register' page with the heading 'Begin your new learning experience by simply filling out this form.' Below the heading is a progress bar with two steps: '1 User Profile' (active) and '2 Additional Fields'. The form fields for Step 1 are: Username (Email) (Required), Email (Required), First Name (Required), Last Name (Required), Password (Required) with a toggle icon, Retype Password (Required) with a toggle icon, Language (Required) with a dropdown arrow, and a link 'Already registered? Sign In'. A red 'NEXT' button is at the bottom right.

The screenshot shows the 'Register' page with the heading 'Begin your new learning experience by simply filling out this form.' Below the heading is a progress bar with two steps: '1 User Profile' (completed with a checkmark) and '2 Additional Fields' (active). The form fields for Step 2 are: Clinic/Practice name (Required), Job Title, Dongle number (Required), Country (Required) with a dropdown arrow, City (Required), and a link 'Already registered? Sign In'. At the bottom are red buttons for 'PREVIOUS' and 'REGISTER'.

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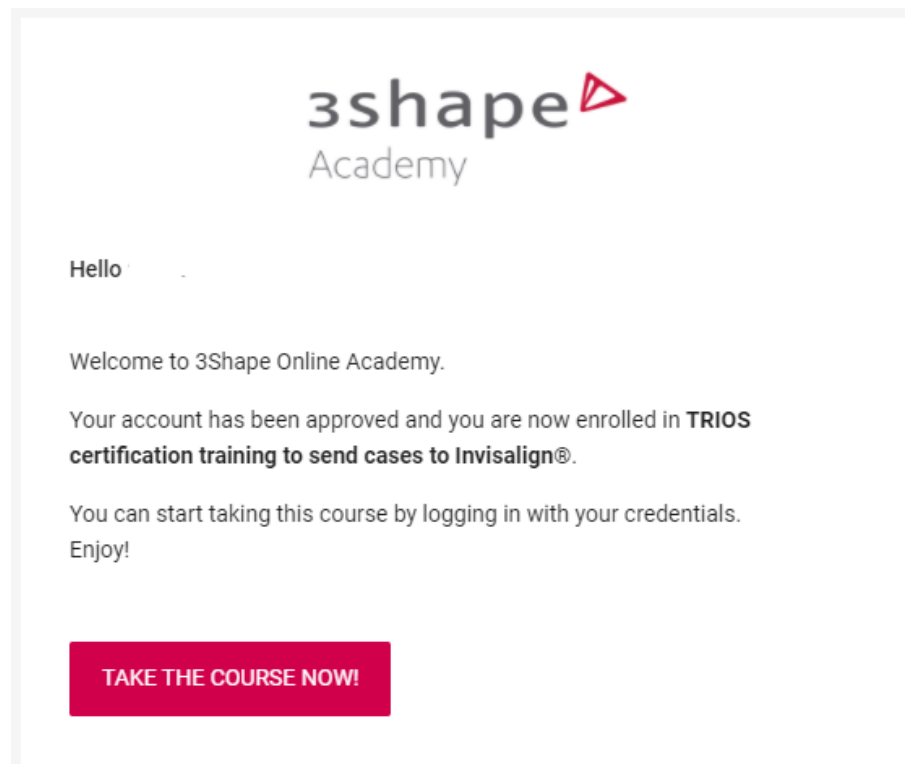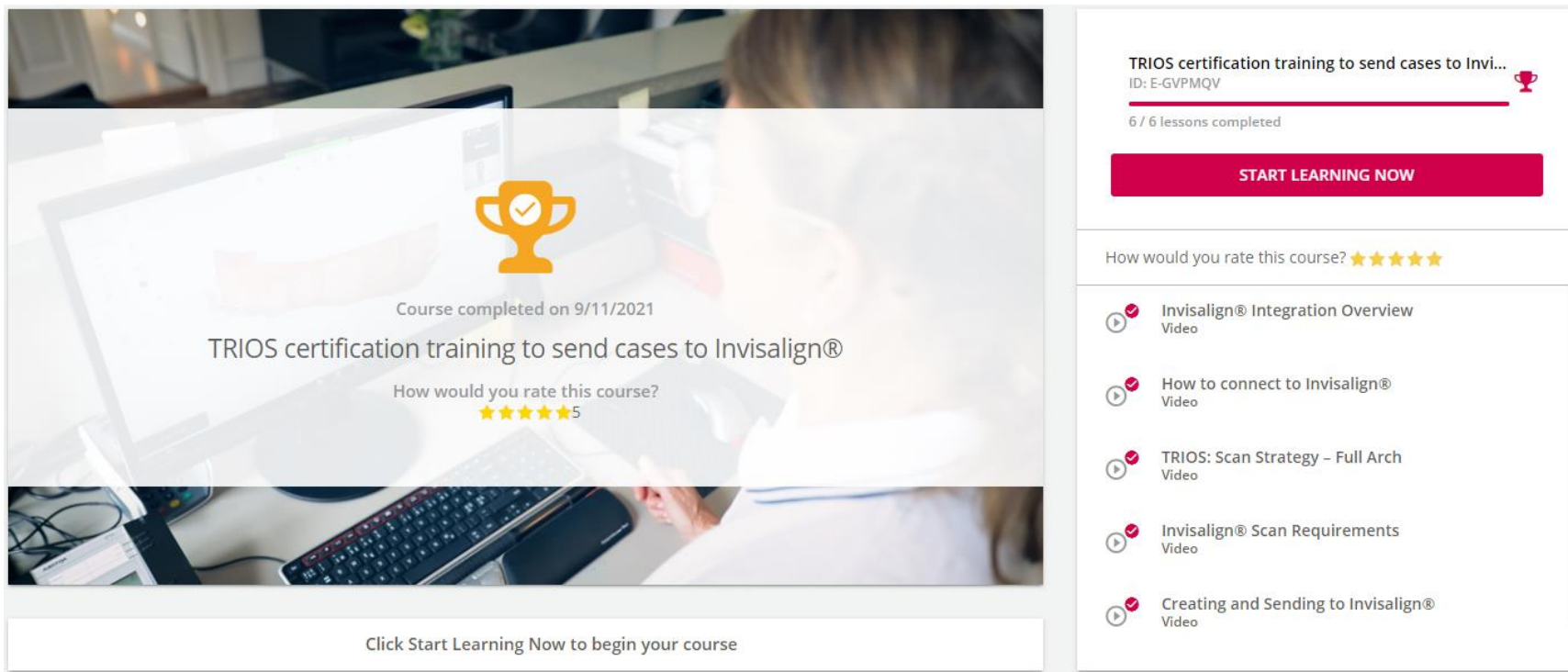
TRIOS certification training to send cases to Invisalign®
ID: E-GVPMQV

6 / 6 lessons completed

START LEARNING NOW

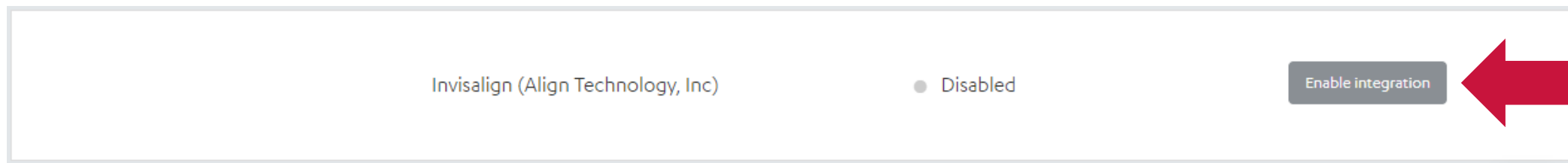
How would you rate this course? ★★★★★

- ▶ Invisalign® Integration Overview Video
- ▶ How to connect to Invisalign® Video
- ▶ TRIOS: Scan Strategy – Full Arch Video
- ▶ Invisalign® Scan Requirements Video
- ▶ Creating and Sending to Invisalign® Video

Click Start Learning Now to begin your course

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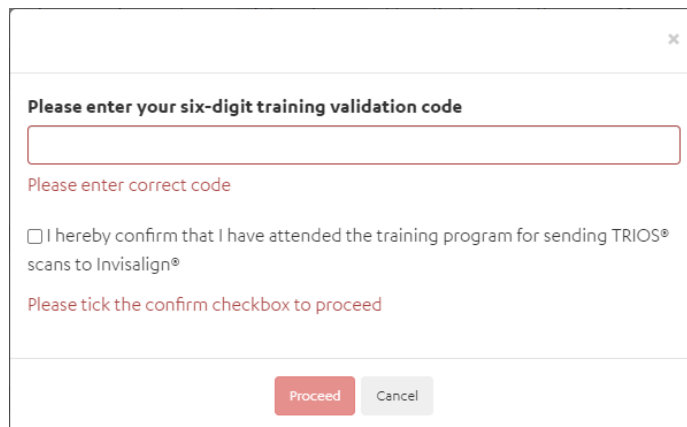
Please enter your six-digit training validation code

Please enter correct code

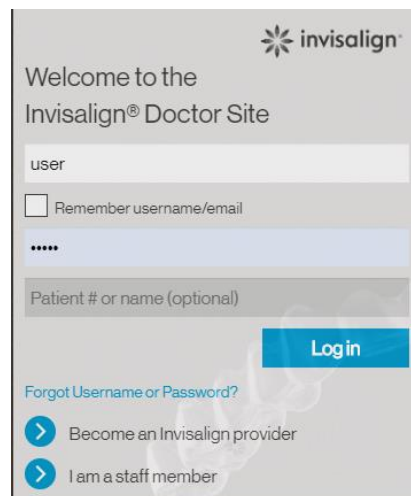
☐ I hereby confirm that I have attended the training program for sending TRIOS® scans to Invisalign®

Please tick the confirm checkbox to proceed

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The screenshot shows the Invisalign Doctor Site login interface. At the top right is the Invisalign logo. Below it, the text reads "Welcome to the Invisalign® Doctor Site". There is a text input field containing "user", followed by a checkbox labeled "Remember username/email". Below that is a password field with masked characters "*****". A text input field for "Patient # or name (optional)" is also present. A blue "Login" button is positioned to the right of the patient field. At the bottom, there is a link "Forgot Username or Password?" and two links with right-pointing arrows: "Become an Invisalign provider" and "I am a staff member".

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